


# QUICK START DIRECTIONS

## Kangaroo™ Connect Enteral Feeding Pump

### PUMP OVERVIEW

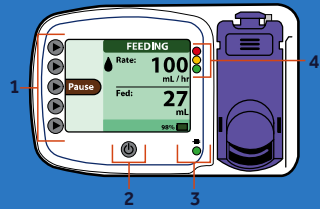

#### 1. Soft keys

Press a key to select the option that appears next to it on the screen.

#### 2. Power button

Press once to power on.  
Press and hold to power off.

#### 3. Power source


A green light shines when the pump is connected to AC power.

#### 4. Pump status


● = Warning alarm  
● = Notice or caution alarm  
● = Standby or feeding

### SET UP THE PUMP


1. Press the Power button.

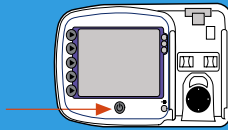

2. Load set by inserting from the bottom and snapping into place.

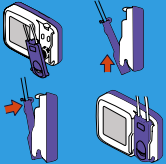


3. Select Prime.

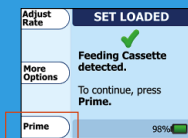


### PRIME THE PUMP


1. Select Priming mode.

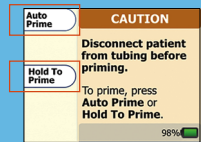

2. If you selected Auto Prime: wait.

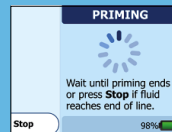


2. If you selected Hold To Prime: Press and hold Prime Feed until fluid reaches end of tubing.

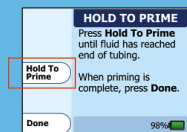


## START A FEEDING


1. Select Clear Settings.

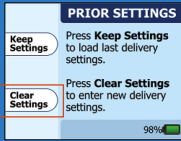


2. Use the plus (+) and minus (-) buttons to enter the rate, then select OK.

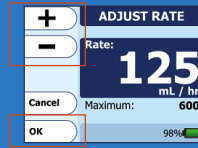


3. When ready to begin feeding, select Start.

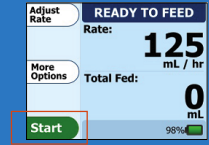

4. To suspend the feeding, select Pause.

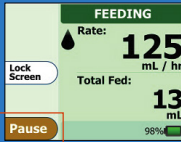


5. To resume feeding, select Continue.

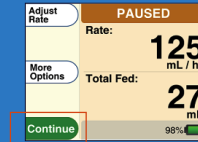


## ENSURE THE CORRECT FLUID LEVEL

Make sure the top of the fluid column is placed 10" (25.4 cm) above the top of the pump.

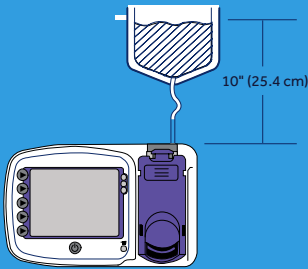

## POWER DOWN THE PUMP

Press and hold the Power button for 3 seconds.

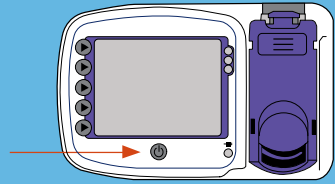

For additional operations or information, see the **Kangaroo™ Connect enteral feeding pump operations manual**

©2016 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 12/2016–H9491 – [WF# 1179205]